




## MAGNETIC MOUNTING PLATE Mounting Instructions

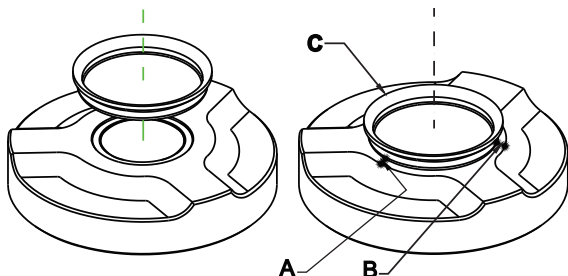

**THE ASSEMBLING DISC MUST BE POSITIONED IN THE PLATE CONCENTRICALLY. BE GUIDED BY THE SIDE MARGINS (A, B, and C), WHICH MUST BE EQUAL.**

The magnetic mounting plates were developed to allow a quick change of the plaster cast in the articulators (without screws), providing a more convenient and quick work, with several cases at the same time. The use of such plates eliminates the need for breaking the casts.

To avoid accumulation of plaster in the magnetic set, the assembling disc must be mounted concentrically in relation to the magnetic plate, according to the guidelines in the figure.

Bio-Art is not responsible for eventual problems or distortions, in case the original assembly discs are not used during the works.

**Note: The magnetic mounting plates are not interchangeable.**



---

Rua Teotônio Vilela, 120 - Jd. Tangará - CEP 13568-000 - São Carlos - SP - Brasil  
Tel. +55 (16) 3371-6502 - Fax +55 (16) 3372-5953  
[www.bioart.com.br](http://www.bioart.com.br)